

Hamilton T1 Non Invasive Set up

<10kgs

- Nasal Mask
- 2 Modes – NCPAP & NCPAP/PC (BIPAP)
- Paediatric tubing
- No flow sensor
- Attach clear pressure line
- Pre use check – complete circuit check
- Start ventilation

*Respiratory alarm turned on –Tempus Pro

>10kgs

- Face mask
- 2 Modes – NIV & NIV-ST (BIPAP)
- Adult tubing
- Attach flow sensor
- Pre use checks as normal – flow sensor
- Start ventilation

*Respiratory alarm turned on –Tempus Pro