

## Turn the pump on

Press **Standby** Check battery status



## Loading the syringe

Flip door down. Pull arm out & to Right




## Correct flange positioning

Load syringe & lift arm back into place.




## Choose syringe brand

Pick correct syringe (use  $\updownarrow$  &  $\leftarrow$  arrows)

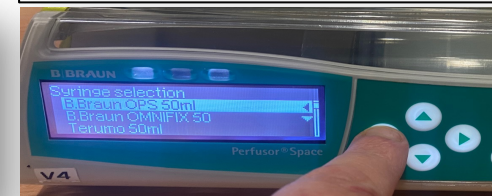


## Access drug library (if line already primed)

no ' $\downarrow$ ' to Prime, yes ' $\uparrow$ ' to drug library

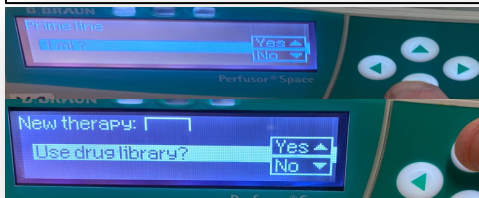


## Select patient weight band

Scroll ' $\updownarrow$ ' & select ' $\leftarrow$ ' Weight Band

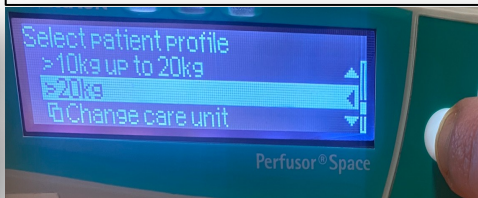


## Select drug category

Scroll  $\updownarrow$  & select category with  $\leftarrow$  arrow

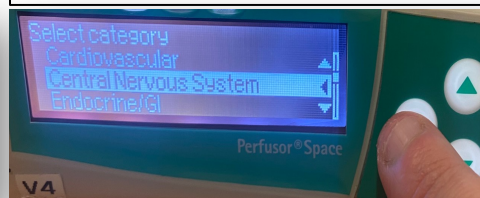


## Find desired drug step

Scroll  $\updownarrow$  & select drug with  $\leftarrow$  arrow

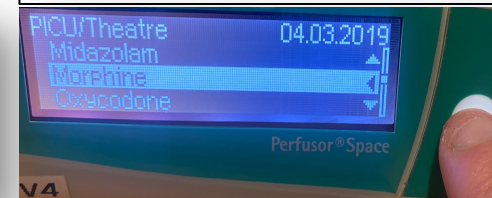


## Enter patient weight

Always in Kg. **Take care with decimals**

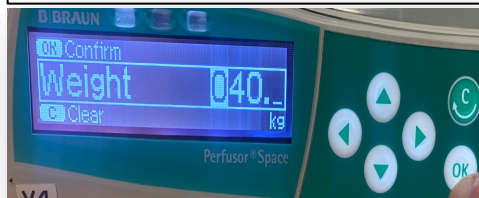


## Preview screen

Start dose visible. To Edit press ' $\leftarrow$ '

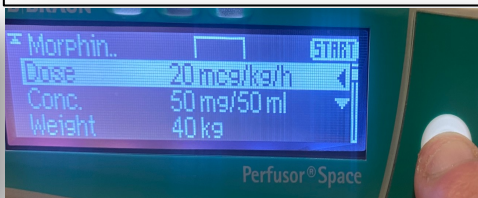


## How to edit dose prior to starting

Edit dose with  $\updownarrow$  arrows. **OK** to confirm

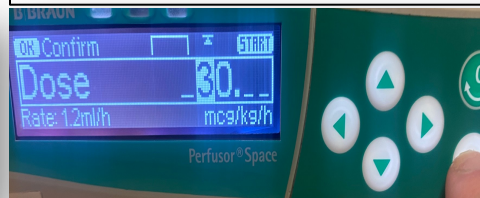

## Commence infusion

Press **start** to commence infusion.

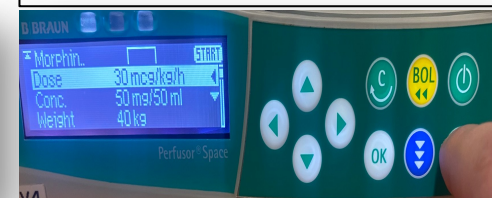


## How to give a bolus (where permitted)

To give bolus, press **BOL** button



## Bolus dose selection

select bolus dose. Upper Limits will apply

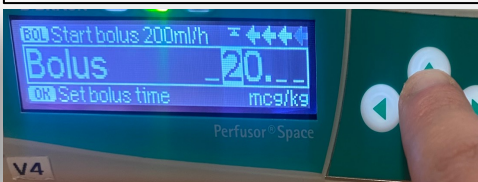


## How to deliver bolus dose

To deliver dose, press **BOL** again



## Normal infusion rate will resume

Once delivered, normal rate will restart

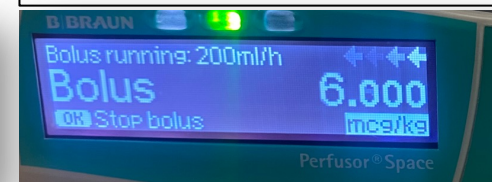

### 'Syringe loaded incorrectly'

Likely syringe flange in wrong location




### Syringe loading correction

Ensure flange is behind the green spring



### Wrong Weight Band/pt. weight/Drug

Wrong drug/wt/conc. identified

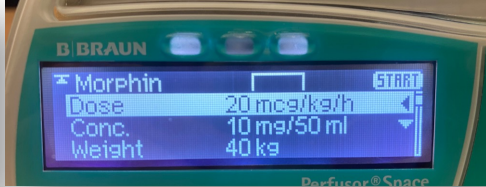


### Correcting wrong Drug / Weight / Band

To re-prescribe, press **C** button




### Correcting wrong Drug/ Weight / Band

Select 'no' ↓ Use last therapy & 'yes' ↑ Use library

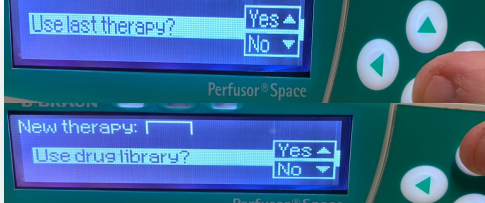


### Upper limit exceeded

May be overridden in some cases by pressing yes ↑. **Should trigger careful review of dose**

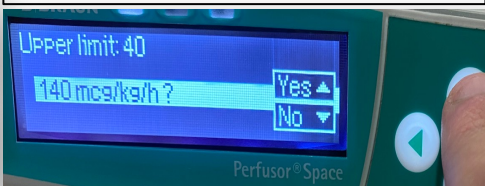

### 'DoseGuard Values have reached limit'

Will appear for either bolus/infusion rate in excess of safety dose. **Cannot be overruled by user.** Lower dose must be selected

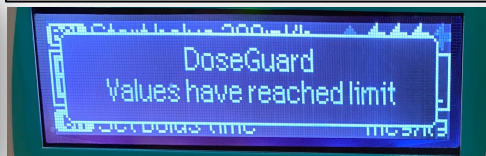


### Changing dose once infusion commenced

← arrow then ↑ & **OK** to select dose



### How to pause pump

Press **STOP** button to pause infusion

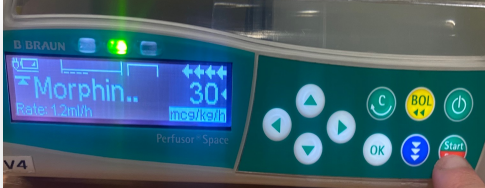

### How to place pump on standby

Press and hold **Standby** button




### How to remove syringe step 1

Gently pull & turn pump arm to the right



### How to remove syringe step 2

Pump will offer a syringe change – Yes ↑

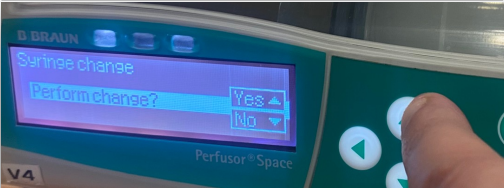

### How to remove syringe step 3

Open driver door and remove syringe



### How to shut down pump correctly

Hold **Standby** button. Arm will retract



### Pump ready for next use

Screen as shown, arm full retracted



### Please ensure pump left charging

Ensure pump is charging before finishing

