

Paediatric Vent Tubing

Equipment: Aerogen neb solo chamber, Neonatal Tee adapter, pink HME filters (x2). Aerogen USB power lead/controller.




STEP 1

Remove HME from circuit. Insert pink HMEs onto both insp. & exp. ports of Hamilton to protect patient. **Only for paed vent circuit**



STEP 2

Connect solo chamber to Tee Adapter & insert upright into the **Blue** inspiratory limb as shown. Connect white power cable to Solo

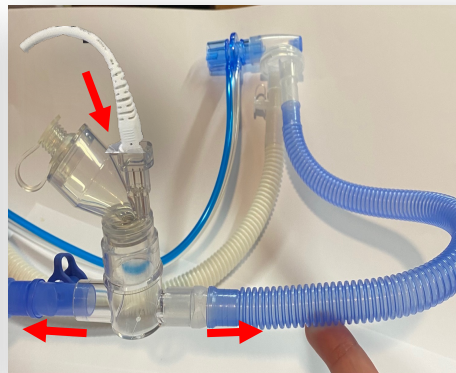

STEP 3a

Insert the Aerogen power lead into the USB port on the side of vent. **Re-site HMEF +/- ETCO2 into circuit after neb & remove HMEs from I/E port to reduce dead space in circuit**




Adult Vent tubing

Equipment: Aerogen neb solo chamber & Adult Tee adapter. Green adapters are kept in the Aerogen bag.



STEP 1

Connect chamber to T. adapter. Connect narrow end of T. to Green piece. Remove ETCO2 from circuit and replace with Aerogen

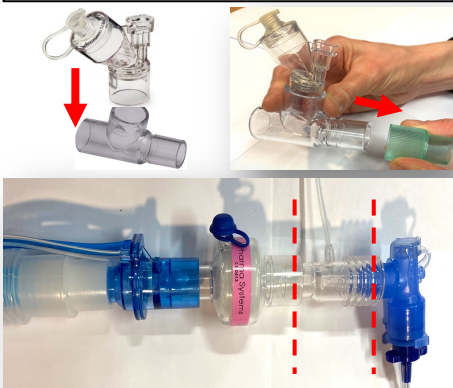

STEP 2

Wide end of Aerogen T. now connected to HMEF and green adapter to the angle piece or ETT directly. Chamber **always** upright



STEP 3b

Place drug in chamber. Press on/off button once for 30 min tx. Press on/off button > 3 seconds to select 6hr (continuous) tx.

