

STEP 1

Warming mattress is located & secured to the top of trolley, warm side facing up. Cover with a cot sheet prior to use.



STEP 2

Back of Unit – **1** Power outlet **2** Power Button **3** Skin sensor probe (not used) **4** Mattress connector



STEP 3

5 Silence **6** Set button **7** Set up/down **8** Orange on/off **9** Set temp **10** Mattress temp



STEP 4

Switch power (**2**) button on at the back of unit to allow charging of the battery and use of unit.



STEP 5

To operate, press orange on/off (**8**) button. Press set button (**6**) & adjust with up/down (**7**) buttons. Press set again to confirm. Battery life: 2 hours



STEP 6

ACRs secured on top of mattress & sheet. Be alert to mental buckles of ACRs, they can get warm. Ensure they are not in direct contact with pt.



STEP 7

Ideally the pt's **core temp** should be monitored whilst mattress is in use. If only skin temp is available please monitor away from mattress.



STEP 8

Clean the mattress after use with alcohol free wipes eg Trionic wipes. Do not place any equipment on mattress whilst not in use.

