




NEVER USE A HOME NIV MASK WITH THE HAMILTON VENTILATOR



STEP 1

Select correct MiniME2 nasal mask using size guide in grab bag. Mask bag contains nasal mask, tubing & soft head strapping.

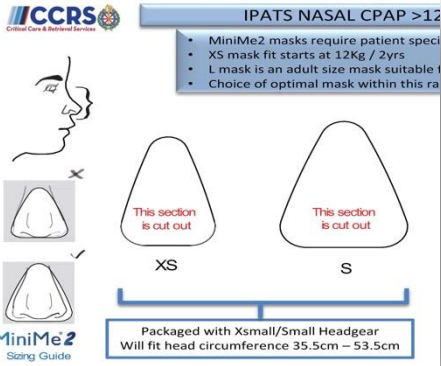

STEP 2

Equipment needed: Tape, Hamilton adult coaxial circuit, Pharma mini port HME filter, MiniME2 nasal mask + hat. **Never use a home-NIV mask. These are vented and not compatible with Hamilton**



STEP 3

Gently push the end of the clear tubing onto the blue mask tubing. A gentle twisting motion is suggested




STEP 4

Set up standard adult co-axial circuit on Hamilton. Attach HME to end of vent tubing & attach nasal mask with tubing onto HME



STEP 5

- Select **Adult/Ped Mode**
- Ensure pre-op checks are finished
- Select NIV & amend settings

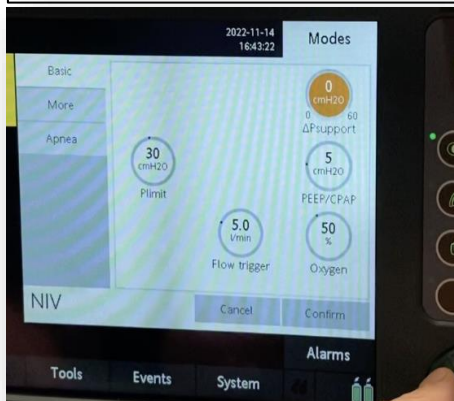


STEP 6

Put mask strapping into position under head. Start ventilation and hold mask over nose while assistant tightens straps



STEP 7

It is essential that the plastic 'bridge' is secured off the face and in a straight line from the mask. Do NOT pull head strap tightly downwards.

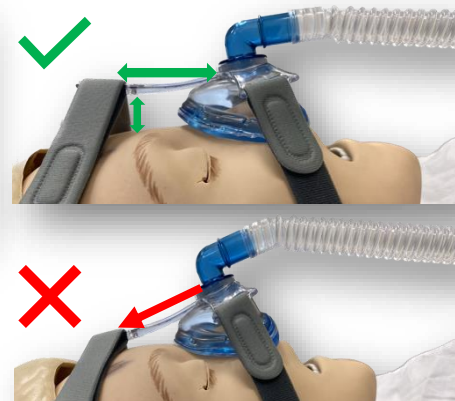

STEP 8

The clear plastic mask piece is malleable and should be moulded as required to fit the patient snugly and adjust for any leaks.

