



STEP 1

Press to turn on. Choose either CG4+ or CG8+ cartridge that has been out of fridge for >30min & is in date.

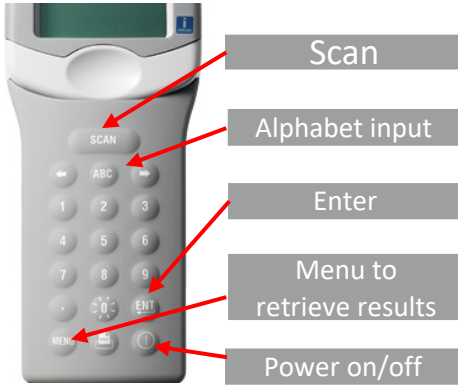


STEP 2

Press **2** *i-STAT cartridge*. Enter **12345** & **ENT**. Enter patients numerical DOB & **ENT**. Follow prompt to scan cartridge in step 3.

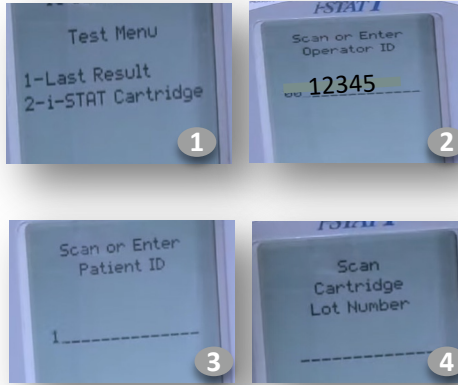


STEP 3

Press **SCAN** & scan barcode on cartridge wrap. Next prompt is to insert cartridge but **STOP**, set device aside & move to step 4.

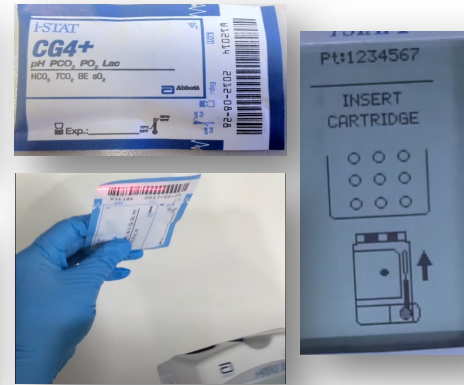

STEP 4

Tear open cartridge pack. Carefully remove cartridge **holding it ONLY at the sides**. Lay on flat surface ready for sample.

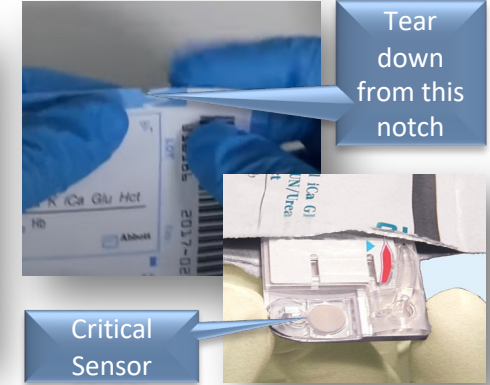

STEP 5

Take cap/syringe gas. Remove first few drops from syringe to remove clots. Min. sample 95µL. Syringe samples easier to handle for iSTAT




STEP 6

Carefully fill well with 2/3 drops of blood to the line as shown. **It is essential not to under/overfill at this point.**



STEP 7

Without touching sensor, lift & fold cover over sample well until it locks. It's normal for blood to move above line during this.

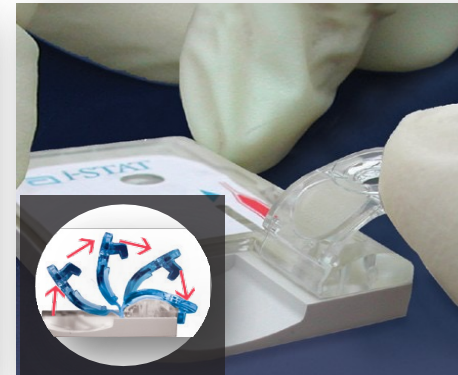

STEP 8

Insert cartridge as shown into the bottom of iStat machine. 2min for results. Remove cartridge & document results.

