


Pack content < 10kgs

- F&P Paediatric ventilator tubing (Please keep pressure line and bag of adaptors)
- Pink expiratory valve
- Blue flow sensor (Not required for NCPAP modes, only added in the event of invasive ventilation)
- CPAP Nasal Tubing (100mms)
- Infant Head gear sizes (x2): 17-22cm, 22-25cms, 25-29cms, 29-36cm, 35-40cm, 40-50cm.
- F&P nasal masks sizes (x2): Small, Medium, Large, extra large
- Measuring Tape


Non Invasive mode set up < 10kgs




- Pressure line attached (supplied with vent tubing) onto **blue** port of ventilator
- **To protect the patient** - place Armstrong **pharma mini HME filter** on to the inspiratory port of the Hamilton ventilator
- **To protect the IPATS team** – place the **Intersurgical Clear Therm green HME** filter on expiratory port

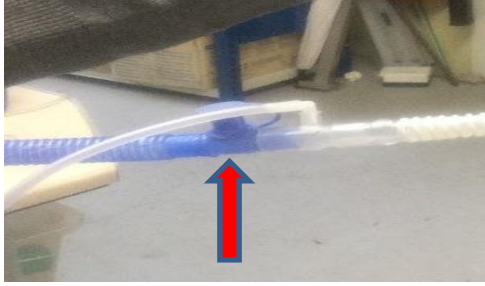

- Clear plastic extension piece with port (supplied with vent tubing) attached to the **inspiratory** limb (blue) of ventilation circuit

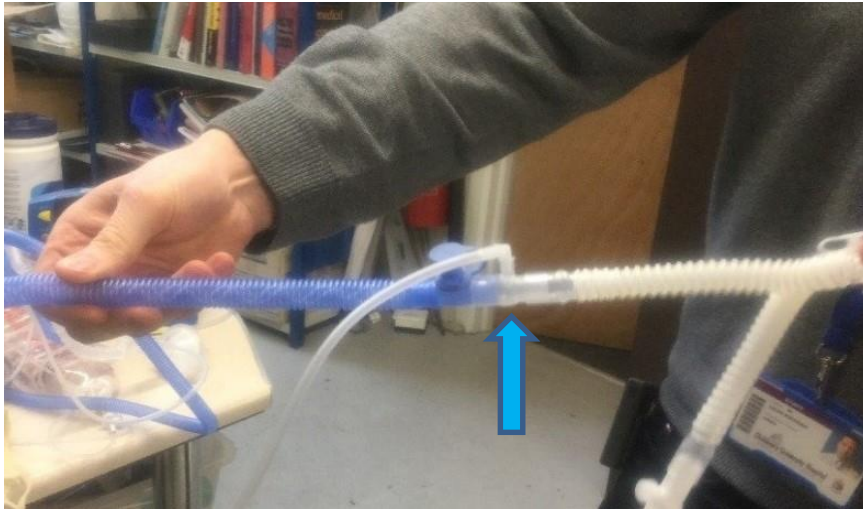

- Pressure line attached onto clear plastic port

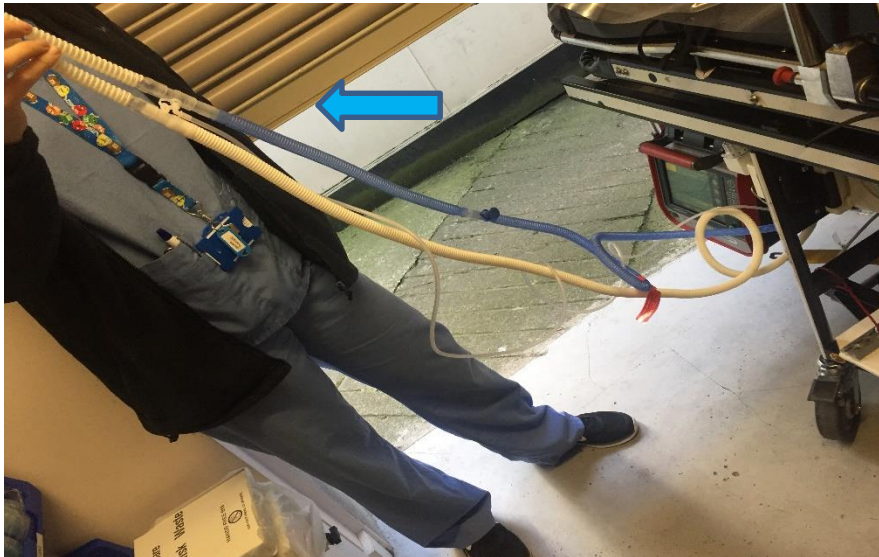


- Complete set up of tubing

< 10kgs (Nasal masks)

- Set up ventilation circuit as above
- Select Neonatal
- Non-invasive options available on the Hamilton screen
 - ✚ **NCAP**
 - ✚ **NCPAP –PC**
 - ✚ **NIV**
 - ✚ **NIV –ST**
- Select ventilation mode **NB: NCPAP and NCPAP –PC** are the **only** 2 modes that can be used for patients < 10kgs
- Enter PEEP (and PC if required), press accept.
- Prior to pressing “Start ventilation”, the pre use checks must be completed.
- Instead of flow sensor check it will require a circuit calibration check. If this is not completed it will alarm “circuit calibration required” when ventilation is started.
- In NCPAP and NCPAP-PC mode – there are “No apnoea detection” and “No disconnection detection” alarms. This is permanently displayed on the screen during use.



- Please ensure Tempus Pro **respiratory alarms** are turned on
- Select Main Menu (**Blue button** on bottom right hand corner) → select Medical Parameter Settings → Select Respiratory Settings
- Ensure Respiratory monitoring is on (green)

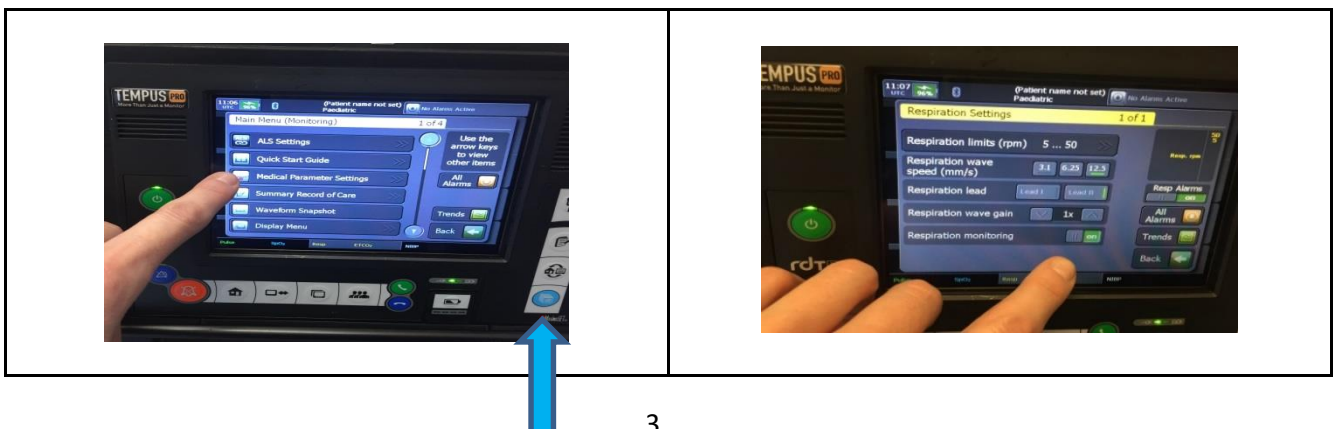

Non Invasive mode set up > 10kgs

Pack content > 10kgs

- Hamilton T1 adult ventilation tubing
- Blue CPAP head strap (x2)
- 22mm non-vented white elbow
- F&P non vented face mask size (x1): small, medium, large

> 10kgs (Face masks)

- Hamilton coaxial ventilation tubing set up as normal, no filters required on inspiratory/expiratory ports – see below
- Complete pre-use checks as normal
- Select appropriate size face mask
- Add HME Filter as below
- Select Paed/Adult
- Non-invasive options available for >10kgs
 - ✚ NIV
 - ✚ NIV –ST
- Select mode required and settings
- Start ventilation
- Please ensure Tempus Pro respiratory alarm is switched on



Normal Hamilton co-axial set up



HME filter in place between facemask & ventilation tubing

