


STEP 1

Open the Ready Heat II pack & expose entire surface to air to begin warming which can take 30min to get to max temp.

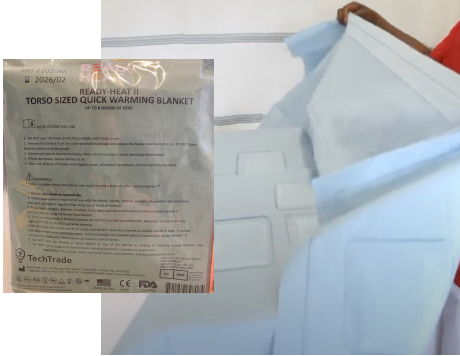

STEP 2

Ensure that patient skin is protected by clothing or a thin sheet. Do NOT apply Ready-Heat directly to skin.



STEP 3

Lay Ready-Heat blanket over sheet. Requires exposure to air to continue to heat. **Do not cover for up to 8min.**




STEP 4

Once heated, further cover can be added as required. If the blanket cools, re-exposing it to air will allow it to heat up again.



STEP 5

Temperature reached is typically 37.8°C. Can reach as high as 43°C. Monitor patient temperature. Stays warm x8hr

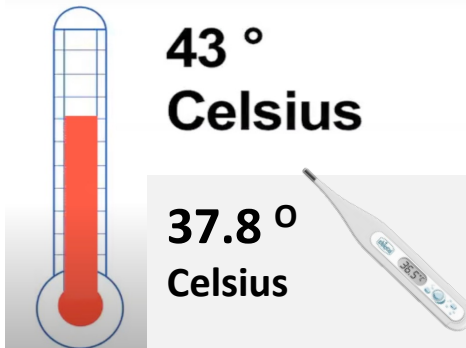

STEP 6

To access patient, do **NOT** fold back blanket. Turn or slide blanket down to access patient.



STEP 7

Do **NOT** lay equipment on top of the blanket. They are prone to overheating & risk focal heating on patient.



STEP 8

Single use item. Dispose in standard bin if uncontaminated & patient not on droplet precautions.

