

NIV Set up	
<u><10kgs</u> • Nasal Mask • 2 Modes – NCPAP & NCPAP/PC (BIPAP) • Paediatric tubing • No flow sensor • Attach clear pressure line • Pre use check – complete circuit check • Start ventilation *Respiratory alarm turned on –Tempus Pro	<u>>10kgs</u> • Face mask • 2 Modes – NIV & NIV-ST (BIPAP) • Adult tubing • Attach flow sensor • Pre use checks as normal – flow sensor • Start ventilation *Respiratory alarm turned on –Tempus Pro